Face Shield

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KEY FEATURES

Lightweight
Multiuse frame
Protects against liquid droplets and splashes
One size

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CE 2777 (EU) 2016/425 ORA 100-B
CE 2777 (EU) 2016/425 ORA 100-A

Description:

Face Shield with clear visor of high quality, durable and lightweight design, and comfortable to wear. Easy to assemble. If cared for and not damaged the face shield frame can be used by individuals multiple times with new visorfilm, this will require proper disinfection with approved machinery for disinfection of medical supplies. Repair of the face shield is NOT permitted

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Protective visor for protection against liquid droplets and splashes. These products are classed as Category III Personal Protective Equipment (PPE) by the European PPE Regulation (EU) 2016/425 and have been shown to comply with this Regulation through limited testing against the Harmonised European Standard EN 166:2001. No protection against other hazards.

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document version: 20201124-1

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The scratched and damaged visorfilm should be replaced. Parts of the visor that come into contact with your skin may lead to allergic reactions. In this case please discontinue use and seek medical advice.

The product does not protect against fast-flying fixed pieces.

Always remember that no item of PPE can provide full protection and care must always be taken while carrying out the risk-related activity.

To optimise protection, it will be necessary to use these products with suitable gloves/gowns/masks. In this case, before carrying out the risk-related activity, consult your supplier or supervisor to ensure that all your protective products are compatible and suitable for your application.

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These products have been tested in accordance with EN 166: 2001 Clauses

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Klausuler: 6 | Design; 7.1.1 | Field of vision; 7.1.2.1 | Refractive powers; 7.1.2.2 | Transmittance; 7.1.2.3 | Diffusion of light; 7.1.3 | Quality of materials; 7.1.5.1 | Stability at elevated temperature; 7.1.7 | Resistance to ignition; 7.2.4 | Droplets and splashes of liquids

Frame and visor markings / Kehyksen ja visiirin merkinnät / Markeringar på ramen och visiret

Frame/Kehys/Ramen

Visor/Visiiri/Visiret

CE 2777 (EU) 2016/425 ORA 100-B

CE 2777 (EU) 2016/425 ORA 100-A

Notified body/Ilmoitettu laitos: Module B and C2 - SATRA Technology Europe Ltd, Bracetown

Business Park, Clonee, D15YN2P, Republic of Ireland (Notified Body 2777).

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Oy Orapac Ab